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KLAPPBARER GEHBARREN

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FOLDABLE PARALLEL BARS

FOLDABLE PARALLEL BARS MODELS

Description	Code
Painted steel handrails	1311010
Wooden handrails	1311020

FISIOTECH SRL

Operating headquarters: Via Toniolo, 44/E-F-G - 31028 Vazzola (TV) - ITALY

Registered office: Via Valdoni, 19 - 31013 Cimavilla di Codognè (TV) - ITALY

Tel. +39 0438470342 - Fax +39 0438478762 - mail : info@fisitech.com - pec : fisitech@emailsec.it

VAT number IT 03765100262 - Rea TV 296956

Table of contents

INTRODUCTION	p. 3
SAFETY – WARRANTY – UNPACKING – DISPOSAL	p. 4
PRODUCT LABEL AND SYMBOLS	p. 5
TECHNICAL SPECIFICATIONS AND MODELS	p. 6
HOW TO USE THE PARALLEL BARS	p. 7
ASSEMBLY INSTRUCTIONS	p. 8
FORBIDDEN ACTIONS	p. 9
WARNINGS – MAINTENANCE – CLEANING	p. 10
INSPECTION RECORD SHEET	p. 11

INTRODUCTION

The purpose of this manual is to provide users and operators with the necessary information to use the parallel bars properly, autonomously, and safely.

This manual contains information on the technical features, operation, maintenance, spare parts and security of the equipment.

This manual must always be carried with the equipment and stored with care so that the readability is not compromised.

In case of transfer of use to a third party, this manual must be transferred as well, along with the other documents, as provided by EU Regulation 2017/745.

Intended use

The parallel bars allow performing exercises for gait rehabilitation safely. They are made of powder-coated steel. The handrails are made either of steel or wood and are adjustable in height.

Manufacturer's details

FISIOTECH SRL

Operating headquarters: Via Toniolo, 44/E-F-G

31028 VAZZOLA (TV) - Italy

Tel. +39 0438 470342

Fax +39 0438 478762

info@fisitech.com

Storing

In case of storing, the following environmental conditions must be ensured: relative humidity 10% ÷ 90 % - temperature -10°C ÷ +50°C – Clean place.

WARNING! Thoroughly read the manual before use.

FISIOTECH SRL refuses any responsibility and is not liable in case any damage to people and/or goods caused by the improper use of the parallel bars, by negligent or not-qualified staff or by non compliance with these instructions occurs.

SAFETY

WARNING! Do not use the the parallel bars:

- On unstable floors or near inclined floors, steps, etc.
- If they have been tampered with or modified.
- If you have not adjusted and tightened Knobs/ratchet handles of the handrails

WARNING!

- Do not adjust the handrails when the patient is using the parallel bars.
- The sliding and moving mechanical parts may harm the user.
- The wooden handrails may catch fire if exposed to open flames.
- Do not spill liquids, as the water accumulation may damage the platform.

WARRANTY

The warranty of the parallel bars covers only the manufacturing defects for 24 months starting from the date of delivery. The warranty does not cover: consumables and items of normal wear, breakdowns caused by any impact and/or fall, wrong or improper use, use by non qualified personnel. The warranty is void if the device has been tempered with. Warranty repairs are carried out at our operational headquarters and prior authorization. The products to be repaired under warranty must be shipped to the manufacturer freight prepaid and with the original packaging. A copy of the invoice and of the transport document has to be sent along with a written request in order for the products to be repaired.

UNPACKING

After placing the pallet containing the couch on a suitable surface cut the plastic straps securing the pallet to the parallel bars and closing the box.

WARNING! Be careful not to cut your hands while handling cutters, scissors or other cutting items. The use of rounded edge scissors is advised to avoid harming your hands and the the equipment. Save packaging for a possible return to the manufacturer.

DISPOSAL


This symbol on a product means that it should not be thrown with regular waste, but disposed of following the laws on recycling/disposal in force in your country/region.

WARNING! The waste must be recycled. It should not be thrown with regular waste, but disposed of following the laws on recycling in force in your country/region.

PRODUCT LABEL – SYMBOLS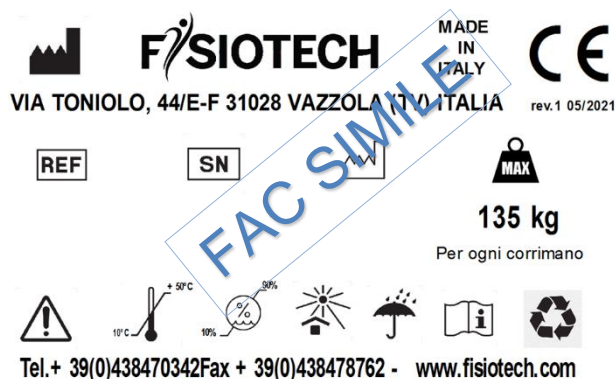


	Manufacturer
	Product Code
	Serial Number
	Date of manufacture
	Max working load
	Warning
	Temperature limit
	Humidity limitation
	Keep away from sunlight and heat sources
	Keep away from rain
	Read the instructions
	Differentiated collection

TECHNICAL SPECIFICATIONS PARALLEL BARS


- A. Anatomic handrails, dimensions 53x43. Wooden or steel handrails, depending on the model.
- B. Textured RAL 9001 powder-coated steel access ramps.
- C. Textured RAL 9001 powder-coated steel tube diameter 48 mm uprights.
- D. Zinc-plated steel tube diameter 42 mm sliding tubes.

Length	300 cm
Handrails width	60 cm
Base width	70 cm
Code 1311010 Height adjustment steel handrails	From 77 to 107 cm
Code 1311020 Height adjustment wooden handrails	From 75 to 105 cm
Single handrail maximum load	135 kg

Code 131010 Foldable parallel bars with steel handrails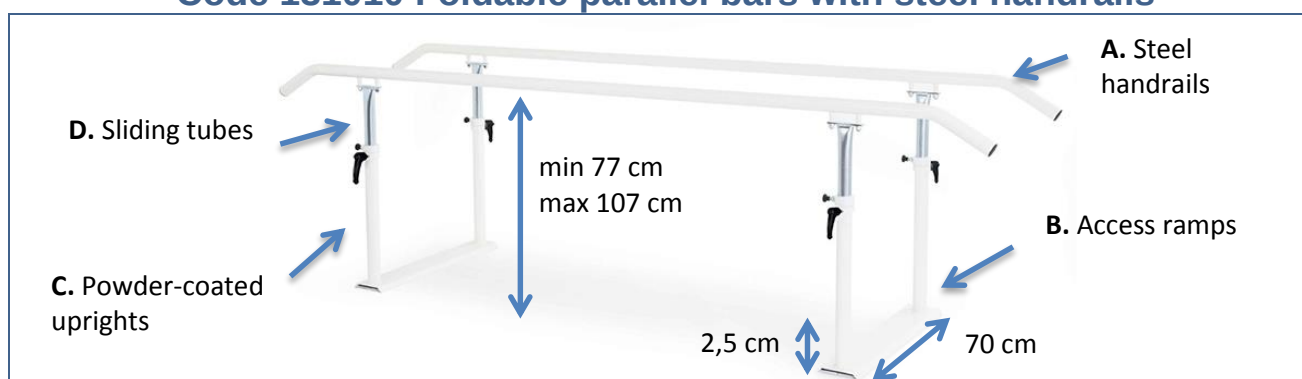


Code 131020 Foldable parallel bars with wooden handrails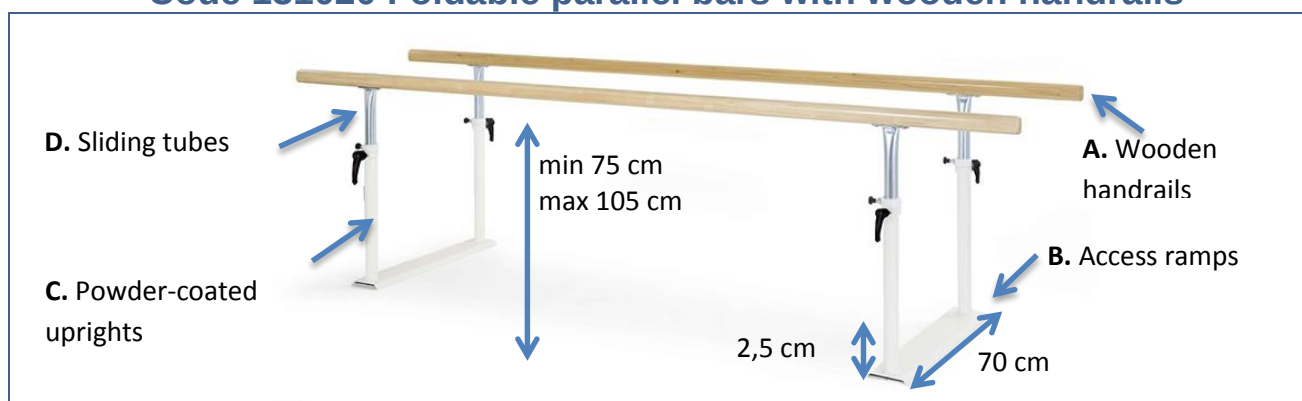

HOW TO USE THE PARALLEL BARS

- In order to adjust the height of the handrails unscrew the knobs/ratchet handles (3), lift or lower the zinc-plated sliding bars (2) moving the spring (4). **Warning!** The handrails must have the same height. **Warning!** When the height has been adjusted, fasten the Knobs/ratchet handles (3) before use.

Height adjustment of the handrails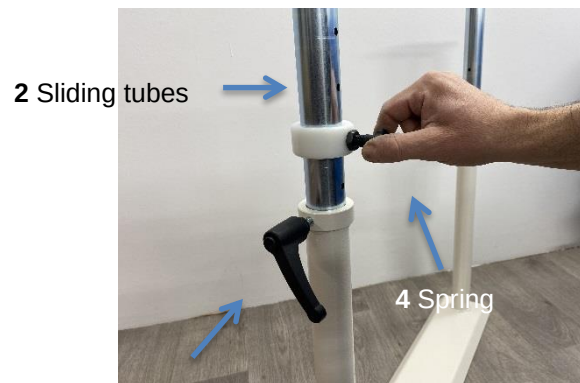

3 Knobs/ratchet handles


WARNING

- Do not adjust the handrails when the patient is using the stairs.
- When the height has been adjusted, fasten the knobs/ratchet handles (3) before use.

ASSEMBLY INSTRUCTIONS

- Unpack all the parts of the parallel bar (see Unpacking page 4).
- Place the bases (5) on the floor as indicated in the image.
- Insert the zinc-plated sliding tubes (2) in the powder-coated uprights (5). The sliding tubes have the position rings and the spring (4) on them. **Warning!** The handrails must have the same height.
- Assembly the handrails (1) on the sliding tubes (2) using the washers (6) and the nuts (7). Make sure the nuts are tightened.
- Warning** When you reach the desired height, tighten the knobs/ratchet handles. Now the parallel bar is ready for use.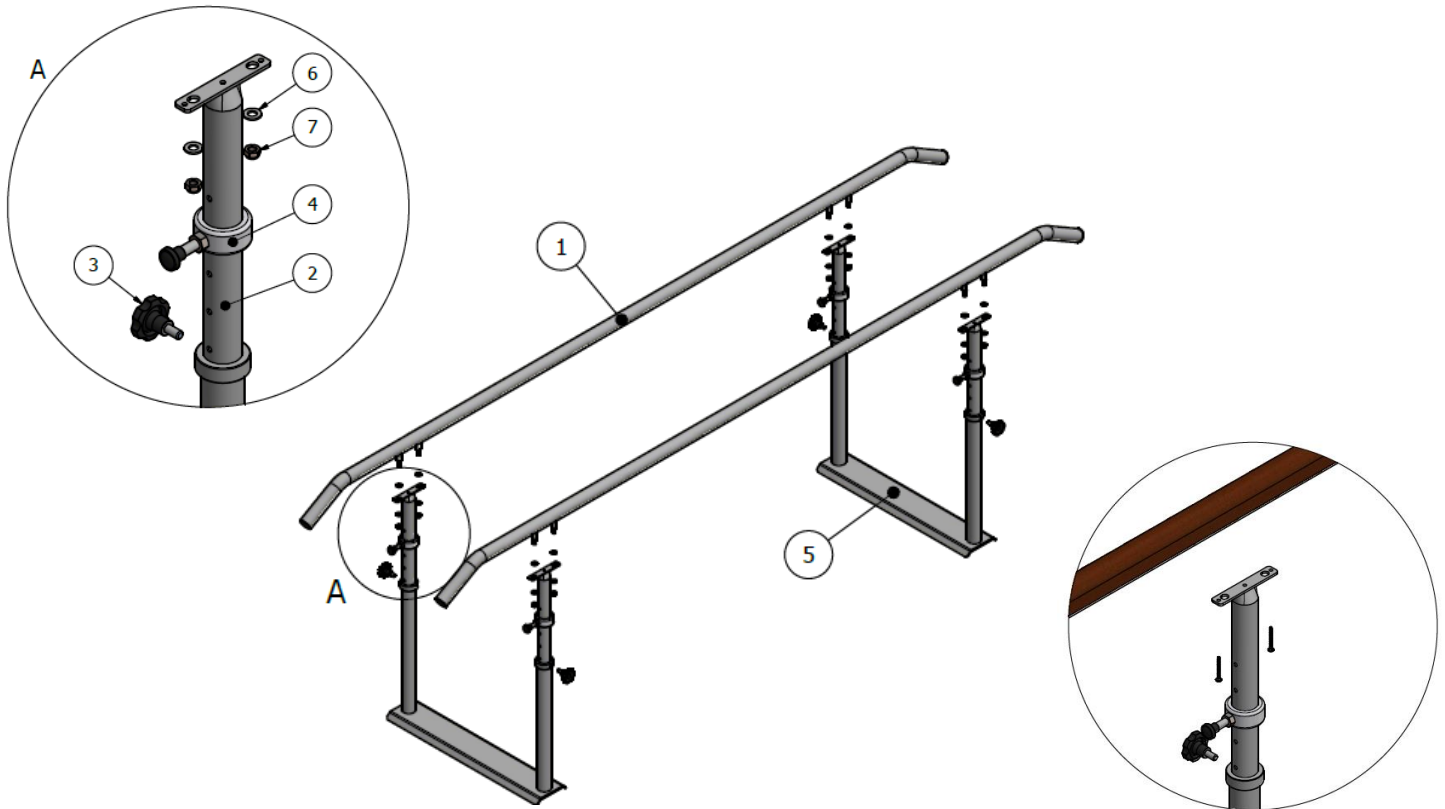

HOW TO FOLD THE PARALLEL BAR

- Unscrew the ratchet handles (3).
-

WARNING

WARNING! The professional should never leave the patient unattended to prevent falls. The manufacturer is not liable for any damage to people and/or goods caused by negligent and/or not-qualified personnel.

MAINTENANCE

Maintenance must be carried out by a specialist, an electrician or someone who is familiar with the content of this manual.

Check the safety conditions of the device after any incident such as falls, violent impacts, spilling of liquids, etc. and any time you deem it necessary.

WARNING!

- In case of safety hazard, the device must be unplugged and is not to be used until safety is restored.

MAINTENANCE	WHAT TO DO
The parallel bars do not need a particular maintenance. We recommend you follow these steps:	Check the tightening of screws and bolts, if the knobs are stripped, the absence of structural damages and deformations every 3 months or after heavy usage.

CLEANING

CLEANING THE PARALLEL BARS: Dust with a dry or at most a lightly damp non-abrasive cloth and let dry.

CLEANING THE HANDRAILS: Dust with a dry or at most a lightly damp non-abrasive cloth and let dry.

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