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Kurt-Georg-Kiesinger-Allee 3 - 53175 Bonn

<https://www2.bfarm.de/medprod/mpsv/>

GB | Elbow bandage, made from recycled plastic

1 | Delivery

1 x Bandage

Read the instructions for use completely and carefully!

2 | Intended purpose

The bandage supports and stabilizes. Targeted pressure on tendons and muscles can relieve pain and relieve the musculoskeletal system. The aim is to reduce the use of painkillers and increase the patient's mobility.

- supports the muscles in case of joint problems, ligament and muscle weakness

- in case of slight twists and sprain
- in case of overload pain
- in case of tendonitis

For private use

3 | Contraindications / Not suitable

In the case of the following medical conditions, it is only recommended to wear this article after consultation with your doctor:

- Skin diseases/injuries in the affected part of the body, especially in inflammatory symptoms such as swelling, redness and overheating
- Allergic reactions to the material
- Sensory and movement disorders, e.g. in the case of diabetes
- Open wounds or internal injuries
- For small children or persons who can not be noticeable when the product sits uncomfortably
- Lymphatic runoff disorders or unclear soft tissue swelling even outside the scope of application.

4 | Notes on safe use of the product

Side effects affecting the whole organism are not yet known. Proper application is required. In case of improper use, product liability is excluded.

- If the bandage is too tight or uncomfortable, remove it immediately!
- Used by the times by private users.
- If your symptoms worsen or you notice unusual changes, consult your doctor immediately.

Do not use: While sleeping

5 | Application

Fig. 1: Put the bandage on below the elbow joint, wrap it around the arm with tension and fix it with the hook and loop closure. Press your arm against your stomach to keep the bandage in place. **Fig. 2+3:** Fix the three elastic bands over the pain point. The stabilization can be adjusted by individual traction.

6 | Cleaning

Close hook and loop fasteners during washing to avoid damaging other items of laundry. Cleaning is done by household laundry, according to care instructions.

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